One Component Self-Etching Light-Cured Dental Adhesive

# TOKUYAMA BOND FORCE

REF AD01

## ENGLISH

Read all information, precautions and notes before using.

### 1. PRODUCT DESCRIPTION AND GENERAL INFORMATION

- TOKUYAMA BOND FORCE is a one-component, one-coat application, self-etching, light-cured, fluoride releasing dental adhesive system that has excellent adhesive properties and excellent marginal integrity toward cut/uncut enamel and dentin without requiring separate etching or priming steps when used in combination with light- or dual-cured composite materials. Its excellent marginal integrity to uncut enamel will enhance esthetic restorations.
- A light-curing unit having the camphorquinone (CQ) wavelength range (peak: 470 nm, spectrum: 400 to 500 nm) can be used for curing TOKUYAMA BOND FORCE.
- TOKUYAMA BOND FORCE contains Phosphoric acid monomer, Bisphenol A di(2-hydroxy propoxy) dimethacrylate (Bis-GMA), Triethylene glycol dimethacrylate, 2-Hydroxyethyl methacrylate (HEMA), Camphorquinone, alcohol and purified water. The pH level immediately after dispensing is approximately 2.3.
- TOKUYAMA BOND FORCE is available in the following packages; Bottle, Pen-Type and Unit Dose. Please check with your local distributor.

### ■INDICATIONS

Bonding of light- or dual-cured composite material to:

- cut/uncut enamel,
- cut/uncut dentin,
- fractured porcelain/composite for repair.

### ■CONTRAINDICATIONS

TOKUYAMA BOND FORCE contains methacrylic monomers, organic solvents and acids. DO NOT use TOKUYAMA BOND FORCE for patients allergic to or hypersensitive to methacrylic monomers, related monomers, organic solvents and acids.

### ■PRECAUTIONS

- DO NOT use TOKUYAMA BOND FORCE for any purpose other than those listed in these instructions. Use TOKUYAMA BOND FORCE only as directed herein.
- TOKUYAMA BOND FORCE is designed for sale and use by licensed dental care professionals only. It is not designed for sale nor is it suitable for use by non-dental care professionals.
- DO NOT use TOKUYAMA BOND FORCE if the safety seals are broken or appear to have been tampered with.
- If TOKUYAMA BOND FORCE causes an allergic reaction or oversensitivity, discontinue its use immediately.
- Use examination gloves (plastic, vinyl or latex) at all times when handling TOKUYAMA BOND FORCE to avoid the possibility of allergic reactions from methacrylic monomers. Note: Certain substances/materials may penetrate through examination gloves. If TOKUYAMA BOND FORCE comes in contact with the examination gloves, remove and dispose of the gloves, and wash hands thoroughly with water as soon as possible.
- Avoid contact of TOKUYAMA BOND FORCE with eyes, mucosal membrane, skin and clothing.
  - If TOKUYAMA BOND FORCE comes in contact with the eyes, thoroughly flush eyes with water and immediately contact an ophthalmologist.
  - If TOKUYAMA BOND FORCE comes in contact with the mucosal membrane, wipe the affected area immediately, and thoroughly flush with water after the restoration. Affected areas may whiten from protein coagulation, but such whitening should disappear within 24 hours. If such whitening does not disappear within 24 hours, immediately contact a physician, and the patient should be so advised.
  - If TOKUYAMA BOND FORCE comes into contact with the skin or clothing, immediately saturate the area with alcohol-soaked cotton swab or gauze.
  - Instruct the patient to rinse his mouth immediately after treatment.
- TOKUYAMA BOND FORCE should not be ingested or aspirated. Ingestion or aspiration may cause serious injury.
- To avoid the unintentional ingestion of TOKUYAMA BOND FORCE, do not leave it unsupervised within the reach of patients and children.
- DO NOT expose TOKUYAMA BOND FORCE or its vapor to open flame because TOKUYAMA BOND FORCE is flammable.
- To avoid cross infection, DO NOT reuse the disposable applicator included in the TOKUYAMA BOND FORCE package.
- Clean the dispensing well included in the TOKUYAMA BOND FORCE package thoroughly with alcohol after each use.
- When using a light-curing unit, protective eye shields, glasses or goggles should be worn at all times.
- In case of the Pen,
  - DO NOT push the button when the cap is closed.
  - DO NOT push the button when the nozzle is facing up.
  - DO NOT hold the Pen diagonally or horizontally while dispensing.
  - DO NOT use the Pen directly in the oral cavity.
  - DO NOT dismantle the Pen.

### ■PRECAUTIONS FOR MEDICAMENTS AND MATERIALS

- Some materials and medicaments (hemostatic material) inhibit adhesion of TOKUYAMA BOND FORCE for an extended period even after meticulous cleansing with water. DO NOT USE products which contain:
  - eugenol,
  - hydrogen peroxide,
  - sodium hypochlorite,
  - diammine silver fluoride [molecular formula: Ag(NH<sub>3</sub>)<sub>2</sub>F],
  - phenols such as parachlorophenol, guaiacol, phenol,
  - aluminum chloride,
  - ferric sulfate,
  - aluminum sulfate,
  - epinephrine.

### ■STORAGE

- Store under refrigeration at temperatures between 0 to 10 °C (32 to 50 °F).
- Keep away from heat, direct sunlight, sparks and open flames.
- DO NOT use after the indicated expiration date on the bottle/package.

### ■DISPOSAL

Unused TOKUYAMA BOND FORCE is to be absorbed into an inert absorbent material such as gauze or cotton, and disposed of in accordance with local regulations.

### ■CLINICAL PROCEDURES

TOKUYAMA BOND FORCE bonds to uncut enamel without phosphoric acid treatment. You don't need to etch the prepared cavity margin before applying TOKUYAMA BOND FORCE. Please read the sections on “**Cleaning**, section 1” and “**Rubbing**, section 8” carefully for successful results.

- Cleaning**  
Thoroughly clean the tooth surface with a rubber cup and a fluoride-free paste then rinse with water.
- Isolation**  
The rubber dam is the preferred method of isolation.
- Cavity preparation**  
Prepare the cavity and rinse with water. Add bevels to the enamel margins of anterior preparations (class III, IV, V), as well as chamfers to the margins of posterior preparations (class I, II) because bevels and chamfers assist in erasing demarcations between the cavity margins and the restoration, thereby enhancing both esthetics and retention.
  - In case of porcelain/composite repairs, roughen the surface with a bur or a diamond point to prepare the area for adhesion; apply phosphoric acid for cleaning; rinse thoroughly with water; air dry thoroughly and treat with a silane coupling reagent according to its manufacturer's instructions.

#### 4. Drying

Dry the cavity by using a blotting technique or an air syringe technique.
 

- DO NOT desiccate the vital tooth. Desiccation can lead to post-operative sensitivity.
- The substances listed below, which inhibit curing, should be removed from the tooth surface by thoroughly cleaning the tooth surface with alcohol, citric acid, or the application of phosphoric acid for 2 to 3 seconds before application:
  - 1) Oil mist from handpiece,
  - 2) Saliva, blood and exudates.

#### 5. Pulp Protection

Glass ionomer lining or calcium hydroxide should be applied if the cavity is in close proximity to the pulp. DO NOT USE EUGENOL BASED MATERIALS to protect the pulp as these materials will inhibit curing of TOKUYAMA BOND FORCE.

#### 6. Dispensing

- 6-1. Unit Dose
  - Twist open the Unit Dose container and dip the disposable applicator.
  - When opening the cap, make sure to verify the top and bottom of the container.
  - To avoid cross infection, do not reuse the material on other patients.

#### 6-2. Bottle

Open the bottle cap and dispense one or two drops of adhesive onto the Dispensing Well. Close the bottle cap tightly immediately after dispensing.
 


- Wipe off the excess adhesive on the tip of nozzle before closing.
- Do not mix the adhesive with other brands of primers or adhesives.

#### 6-3. Pen

Open the cap of the Pen and dispense one or two drops of adhesive onto the Dispensing Well by holding the Pen vertically and pushing on button repeatedly until adhesive starts to flow (3 or 2 pushes, This depends on how the button is pushed and the position that the Pen is being held). Close the cap of the Pen tightly immediately after dispensing.
 


- Be sure to hold the Pen vertically when dispensing. Do not hold the Pen diagonally or horizontally while dispensing; this could cause the adhesive to flow back into the nozzle and become contaminated.
- Wipe off the excess adhesive from the tip of the nozzle prior to closing.
- Do not mix the adhesive with other brands of primers or adhesives.

#### ■ How to open and close the Pen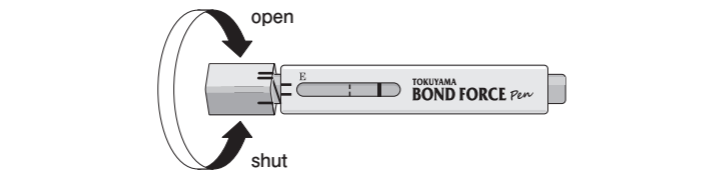

Indicator:

The indicator allows users to monitor the remaining amount of adhesive. Black Line moves closer to the E as the amount of material decreases. When the line nears “E”, this indicates that the Pen is almost empty. This is not an exact reading but only an indication of the remaining material.

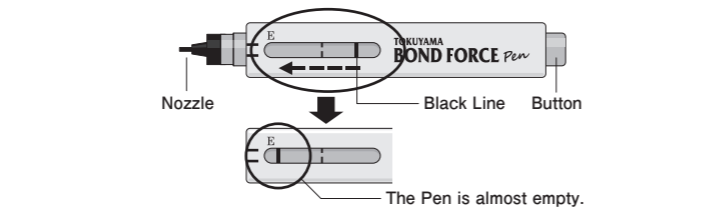#### 7. Application

Apply the adhesive to the cavity walls embracing both enamel and dentin using the disposable applicator. Regarding the prepared cavity margin, the adhesive should be extended just a little into the uncut enamel side. Be sure not to leave out any areas where the adhesive should be applied.

- Protect the dispensed adhesive and the inserted applicator from ambient light before the application.

Unit Dose: keep the applicator into the Unit Dose container.

Bottle, Pen: use a light blocking plate.

- Complete the application within 5 minutes after dispensing because TOKUYAMA BOND FORCE contains a volatile alcohol.

#### 8. Rubbing

Rub the applied adhesive along the demarcation line (beveled or chamfered areas) using the same applicator under light finger pressure for 20 seconds or more, immediately following the application. The surrounding uncut enamel should also be rubbed accordingly.

- The rubbing time does not include the application time. A separate rubbing time is needed to ensure proper adhesion (especially for marginal enamel). In case of multiple restorations, ensure individual rubbing time for each restoration.
- If saliva, blood, or other fluids contaminate the applied/rubbed adhesive, thoroughly rinse the cavity with water, dry and re-apply new adhesive.
- Do not rinse the applied/rubbed adhesive with water except in unintentional contamination.

#### 9. Air Dry

Using an oil-free air/water syringe, apply weak air flow to the adhesive surface, continuing until the runny adhesive stays in the same position without any motion (usually for 5 seconds). Finish by using a strong air flow for 5 seconds or more. Use a vacuum aspirator to prevent spatter of the adhesive.

- If accidental spattering occurs, it may cause the tissue to whiten or possible allergic reaction.
- To prevent spattering, refer to the following tips:
  - 1) To avoid inadvertent strong air;
    - a) Be sure to begin weak air flow outside the mouth,
    - b) Direct the air flow at the adhesive surface.
  - 2) Extending the distance of the air/water syringe from the tooth reduces the air flow. The mirror refraction technique is also useful in reducing the air flow.
  - 3) If the adhesive pools on the cavity floor or the cavosurface angle and is too thick to air thin, blot the excess with a new disposable applicator before weak air application.

#### 10. Light-cure

Light-cure the surface for 10 seconds or more, keeping the curing light tip within a distance of 2 mm from the surface. If the cavity is too large or too distant (e.g. MOD), divide the area into segments and light-cure each segment individually.
 

- Confirm that the light-curing unit has sufficient intensity (>300 mW/cm<sup>2</sup>) before using. Note that using a cracked light guide will lower the intensity.

#### 11. Light-cured Composite

Restore with light-cured composite resin according to its manufacturer's instructions. The overfilled composite resin should be thoroughly finished and polished.
 

- When dual-cured composite resins are placed into a cavity, the first increment must be light-cured using a layering technique.
- Do not use self-cured composite resins here because the Phosphoric acid monomer contained in TOKUYAMA BOND FORCE may interfere with the curing of the self-cured resins, leading to their premature detachment.

The manufacturer of the TOKUYAMA BOND FORCE is not responsible for damage or injury caused by improper use of this product. It is the personal responsibility of the user to ensure that this product is suitable for an appropriate application before use.

Product specifications of the TOKUYAMA BOND FORCE are subject to change without notification. When the product specifications change, the instructions and precautions may change also.

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