

Use this card for pre-operative simulation.

Clinical Guide

PALFIQUE
OMNiCHROMA



PALFIQUE
omniCHROMA
Every Shade, One Choice

5255182003

Additional information —————
To enhance both esthetics and retention, add bevels to the
enamel margins of anterior preparations.

PALFIQUE omniCHROMA

The color of PALFIQUE OMNICHROMA paste is opaque-white before curing. It turns natural in appearance and blends with the color of the surrounding tooth after light curing.

Before light curing



After finishing



— PALFIQUE OMNICHROMA — **BLOCKER**

Use BLOCKER under PALFIQUE OMNICHROMA for the direct restoration of extensive Class III or IV cavities as a thin layer at the lingual cavity wall.

The necessary thickness of PALFIQUE OMNICHROMA BLOCKER depends on the case. 0.5mm is illustrated as a guide, but can be thicker for whiter and opaque teeth, or thinner for chromatic and translucent teeth.

