Resin-based Dental Restorative Materials

ESTELITE FLOW QUICK *High Flow*

ENGLISH

Carefully read all instructions, precautions and notes before using ESTELITE FLOW QUICK High Flow.

■PRODUCT DESCRIPTION AND GENERAL INFORMATION

- ESTELITE FLOW QUICK High Flow is a low viscosity, high flow, light-cured, radiopaque composite resin. Direct placement of ESTELITE FLOW QUICK High Flow into prepared cavity makes handling easy.
- ESTELITE FLOW QUICK High Flow implements Radical-Amplified Photopolymerization initiator technology (RAP technology). Curing time for ESTELITE FLOW QUICK High Flow can be shortened by one-third using a light curing unit with a wavelength range of 400 to 500 nm (peak 470 nm) compared to our conventional products. Please see the table depicting the relationship between curing time and increment depth (■INDICATION FOR FILLING AND CURING).
- RAP technology facilitates a longer working time (90 sec.) for ESTELITE FLOW QUICK High Flow.
- ESTELITE FLOW QUICK High Flow contains 68% by weight (49% by volume) of silica-zirconia filler. Every inorganic filler contained in ESTELITE FLOW QUICK High Flow is a spherical filler (mean particle size: 300 nm, particle size range 40 to 600 nm) that enables excellent gloss retention and wear resistance. The monomer matrix contains Bisphenol A di(2-hydroxy propoxy) dimethacrylate (Bis-GMA) and Triethylene glycol dimethacrylate (TEGDMA).

■SHADE

A1, A2, A3 and OPA2.

- ESTELITE SIGMA QUICK SHADE GUIDE provided for ESTELITE SIGMA QUICK can also be utilized with the ESTELITE FLOW QUICK High Flow. All shades correspond to ESTELITE SIGMA QUICK shades.

- OPA2 is designed for masking slight internal stain or reconstructing a highly opaque tooth.

- Please check with your local distributor for available shades.

■INDICATIONS

- Direct anterior and posterior restorations (particularly for small/shallow/tunnel shaped cavities)

- Cavity lining

- Blocking out cavity undercuts before fabrication indirect restorations

- Repair of porcelain/composite

■CONTRAINDICATIONS

ESTELITE FLOW QUICK High Flow contains methacrylic monomers. DO NOT use ESTELITE FLOW QUICK High Flow for patients allergic to or hypersensitive to methacrylic and related monomers.

■PRECAUTIONS

- DO NOT use ESTELITE FLOW QUICK High Flow for any purposes other than those listed in these instructions. Use ESTELITE FLOW QUICK High Flow only as directed herein.
- ESTELITE FLOW QUICK High Flow is designed for sale and use by licensed dental care professionals only. It is not designed for sale nor use by non-dental care professionals.
- DO NOT use ESTELITE FLOW QUICK High Flow if the safety seals are broken or appear to have been tampered with.
- If ESTELITE FLOW QUICK High Flow causes an allergic reaction or hypersensitivity, discontinue immediately.
- Use examination gloves (plastic, vinyl or latex) at all times when handling ESTELITE FLOW QUICK High Flow to avoid the possibility of allergic reactions from methacrylic monomers. Note: Certain substances/materials may penetrate through examination gloves. If ESTELITE FLOW QUICK High Flow comes in contact with the examination gloves, remove and dispose of the gloves, and wash hands thoroughly with water as soon as possible.
- Avoid contact of ESTELITE FLOW QUICK High Flow with eyes, mucosal membrane, skin and clothing.
 - If ESTELITE FLOW QUICK High Flow comes in contact with the eyes, thoroughly flush eyes with water and immediately contact an ophthalmologist.
 - If ESTELITE FLOW QUICK High Flow comes in contact with the mucosal membrane, wipe the affected area immediately, and thoroughly flush with water after the restoration is completed.
 - If ESTELITE FLOW QUICK High Flow comes into contact with the skin or clothing, immediately saturate the area with an alcohol soaked cotton swab or gauze. Instruct the patient to rinse his mouth immediately after treatment.
- ESTELITE FLOW QUICK High Flow should not be ingested or aspirated. Ingestion or aspiration may cause serious injury.
- To avoid the unintentional ingestion of ESTELITE FLOW QUICK High Flow, do not leave it unsupervised within the reach of patients and children.
- Clean the placement instruments with alcohol after use.
- When using a light-curing unit, protective eye shields, glasses or goggles should be worn.

■INDICATIONS FOR FILLING AND CURING

Be sure to light-cure ESTELITE FLOW QUICK High Flow extraorally and check the time needed for routine handling of ESTELITE FLOW QUICK High Flow with your light-curing unit before performing the bonding procedure. The following table summarizes the relationship between curing time and increment depths.

Relationship between curing time and increment depth:

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- Increment depth was determined on the basis of test results performed in accordance with section 7.10 "depth of cure" of ISO4049: 2000.

■STORAGE

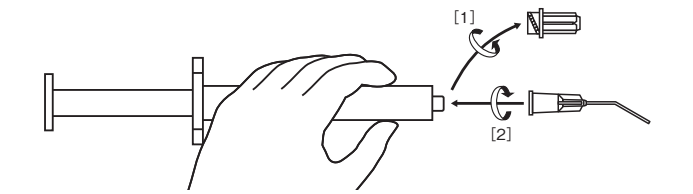
- Store ESTELITE FLOW QUICK High Flow at a temperature between 0 – 25 °C (32 – 77 °F).
- AVOID direct exposure to light and heat.
- DO NOT use ESTELITE FLOW QUICK High Flow after the indicated date of expiration on the syringe/package.

■DISPOSAL

To safely dispose of excess ESTELITE FLOW QUICK High Flow, extrude unused portion from the syringe and light-cure before disposal.

■ATTACHING THE METAL SYRINGE TIP

- Hold the nozzle and turn the cap counterclockwise for removal. [1]
- Hold the nozzle and place the Metal Syringe Tip onto it. Turn the Metal Syringe Tip clockwise until it is securely locked. [2]

- Accessories and Metal Syringe Tip are exclusively designed for ESTELITE FLOW QUICK High Flow. Avoid using Syringe Tips from other brands with ESTELITE FLOW QUICK High Flow.

- Metal Syringe Tips are also sold separately.

- Use gloves when attaching the Metal Syringe Tip.

■CLINICAL PROCEDURE

- Cleaning**

Thoroughly clean the tooth surface with a rubber cup and a fluoride-free paste then rinse with water.
- Shade selection**

Select the appropriate shade of ESTELITE FLOW QUICK High Flow.
- Isolation**

A rubber dam is the preferred method of isolation.
- Cavity preparation**

Prepare the cavity and rinse with water.
- Pulp protection**

Calcium hydroxide should be applied if the cavity is in close proximity to the pulp. DO NOT USE EUGENOL-BASED MATERIALS to protect the pulp as these materials will inhibit curing of ESTELITE FLOW QUICK High Flow. ESTELITE FLOW QUICK High Flow is not indicated for direct pulp capping.

6. Bonding system

Apply light-cured bonding system according manufacturer’s instructions.

- Do not use self-cured bonding systems. We advise that light-cured bonding systems should be used with ESTELITE FLOW QUICK High Flow since other types of bonding systems such as self-cured or dual-cured bonding systems may not provide effective adhesion when used with light-cured composite resins. If you want to use self-cured or dual-cured bonding systems, please confirm that such bonding systems are compatible with the light-cured composite resins being used.
- In case of porcelain/ composite repairs be sure to condition the surface of porcelain/composite with a silane coupling agent BEFORE performing the bonding procedure.

7. Filling and contouring

Cavity can be directly filled from the Metal Syringe Tip, or indirectly filled with an instrument after extruding the paste on the mixing pad. Increments should not exceed the indicated curing depth at a time (please refer to the aforementioned table).

- Immediately after removal from refrigerator, do not apply excessive force to extrude the material from the syringe.
- DO NOT mix ESTELITE FLOW QUICK High Flow with other brands of composite resin to avoid incomplete cure or entrapment of air bubbles. In order to avoid air bubbles entrapment, DO NOT mix ESTELITE FLOW QUICK High Flow with another shade of the paste.
- After extruding the paste, remove and discard the Metal Syringe Tip, wipe the nozzle with gauze and replace the cap immediately.

8. Curing

Light-cure each increment for at least the indicated time (please see aforementioned table).

- If other brands of composite resins are layered over the cured composite, follow their instructions.

9. Finishing

Shape and polish the restoration. For finishing, use fine finishing diamond points. To diminish the demarcation between the composite and the natural tooth, rotate a 12 fluted carbide burr slowly along the margins without water. Use metal finishing strips or vinyl polishing strips for proximal surfaces. For polishing, polish with rubber points or any suitable polishing tools, such as PoGo™ (IDENTISPLY/Caulk), Sof-Lex™ (3M ESPE), Identoflex® HiLuster Dia Polishers (Kerr), or D-FINE™ Hybrid Diamond (COSMODENT, Inc.).

* PoGo, Sof-Lex, Identoflex and D-FINE are not registered trademarks of Tokuyama Dental Corporation.

CAUTION: Federal law restricts this device to sale by or on the order of a dental professional.

IMPORTANT NOTE: The manufacturer is not responsible for damage or injury caused by improper use of this product. It is the personal responsibility of the dental professional to ensure the product is suitable for application before use.

Specifications are subject to be changed without notification. When the product specification changes, the instructions and precautions may change also.

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